# Aesculap

## Econom CL

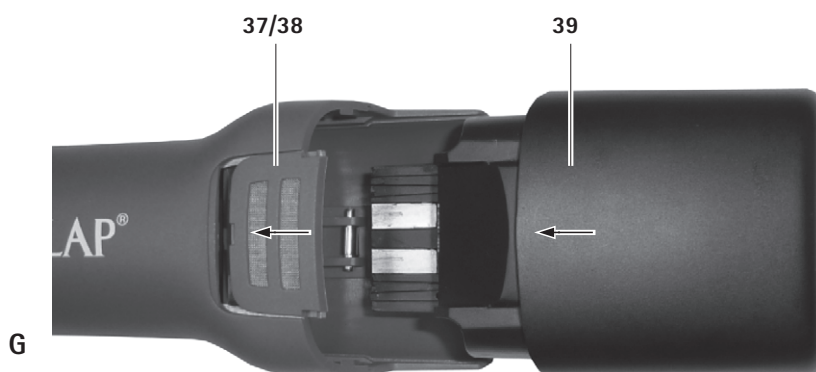
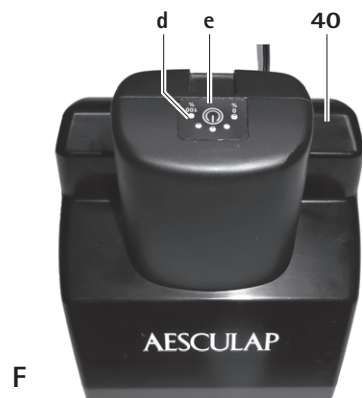
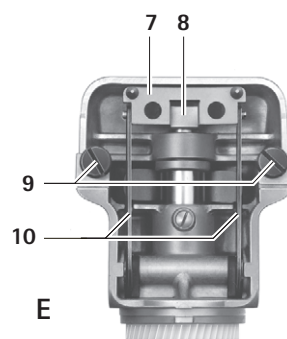
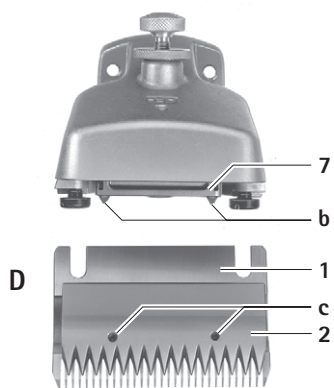
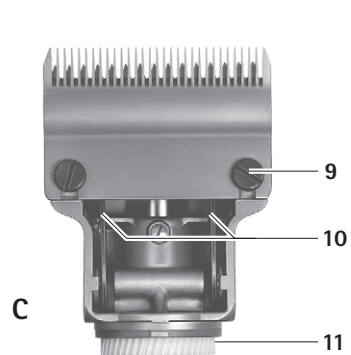
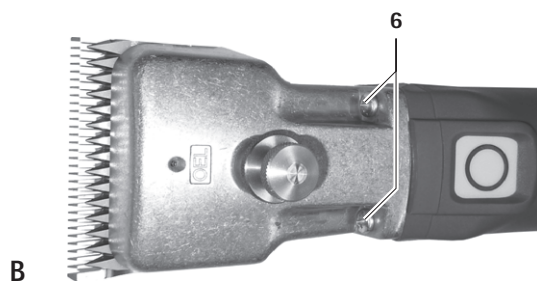
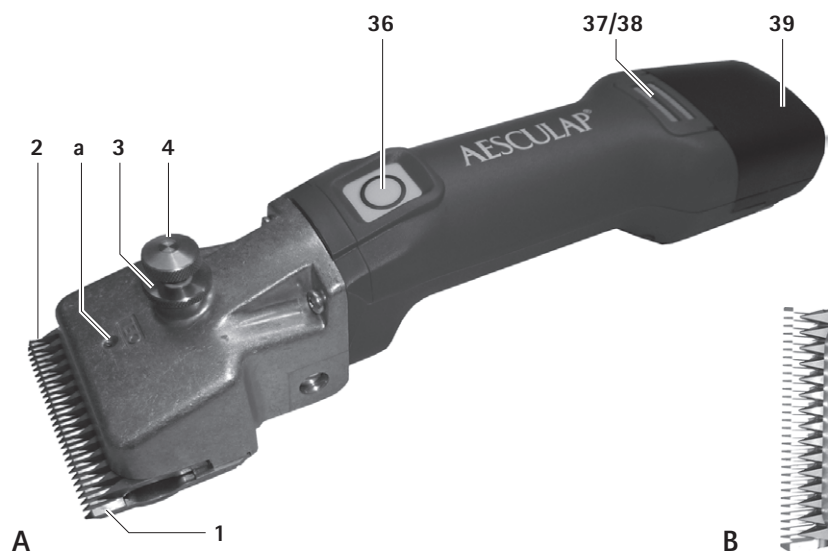
## Econom CL Equipe

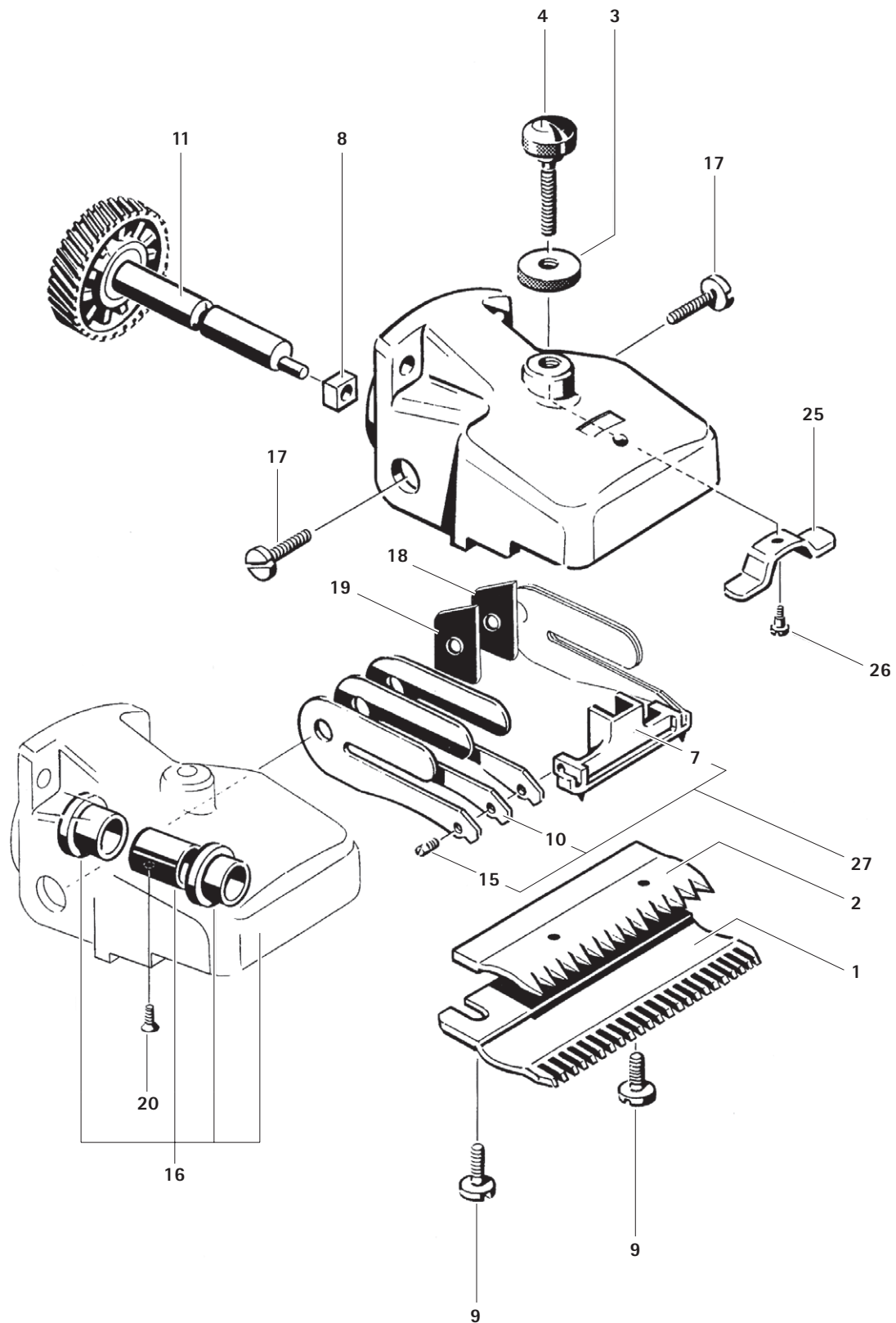
- (GB)** **Instructions for use/Technical description**  
Cattle and horse clipping machine  
Econom CL (GT800)  
Econom CL Set 1 battery (GT804)/2 batteries (GT806)  
Econom CL Equipe (GT810)  
Econom CL Equipe Set 1 battery (GT814)/2 batteries (GT816)
- (USA)** **Instructions for use/Technical description**  
Cattle and horse clipping machine  
Econom CL (GT800)  
Econom CL Set 1 battery (GT804)/2 batteries (GT806)  
Econom CL Equipe (GT810)  
Econom CL Equipe Set 1 battery (GT814)/2 batteries (GT816)
- (D)** **Gebrauchsanweisung/Technische Beschreibung**  
Rinder- und Pferdeschermaschine  
Econom CL (GT800)  
Econom CL Set 1 Akku (GT804)/2 Akkus (GT806)  
Econom CL Equipe (GT810)  
Econom CL Equipe Set 1 Akku (GT814)/2 Akkus (GT816)
- (F)** **Mode d'emploi/Description technique**  
Tondeuse pour la tonte des bovins et chevaux  
Econom CL (GT800)  
Econom CL Set 1 batterie (GT804)/2 batteries (GT806)  
Econom CL Equipe (GT810)  
Econom CL Equipe Set 1 batterie (GT814)/2 batteries (GT816)
- (E)** **Instrucciones de manejo/Descripción técnica**  
Máquina para bovinos y caballos  
Econom CL (GT800)  
Econom CL Set 1 batería (GT804)/2 baterías (GT806)  
Econom CL Equipe (GT810)  
Econom CL Equipe Set 1 batería (GT814)/2 baterías (GT816)
- (I)** **Istruzioni per l'uso/Descrizione tecnica**  
Tosatrice per bovini ed equini  
Econom CL (GT800)  
Econom CL Set 1 accumulatore (GT804)/2 accumulatori (GT806)  
Econom CL Equipe (GT810)  
Econom CL Equipe Set 1 accumulatore (GT814)/2 accumulatori (GT816)
- (NL)** **Gebruiksaanwijzing/Technische Beschrijving**  
Tondeuze voor runderen en paarden  
Econom CL (GT800)  
Econom CL Set 1 accu (GT804)/2 accu's (GT806)  
Econom CL Equipe (GT810)  
Econom CL Equipe Set 1 accu (GT814)/2 accu's (GT816)
- (DK)** **Brugsanvisning/Teknisk Beskrivelse**  
Klippemaskine til kvæg og heste  
Econom CL (GT800)  
Econom CL Set 1 akkumulator (GT804)/2 akkumulatører (GT806)  
Econom CL Equipe (GT810)  
Econom CL Equipe Set 1 akkumulator (GT814)/2 akkumulatører (GT816)
- (N)** **Bruksanvisning/Teknisk Beskrivelse**  
Klippemaskin til storfe og hest  
Econom CL (GT800)  
Econom CL Set 1 batteri (GT804)/2 batterier (GT806)  
Econom CL Equipe (GT810)  
Econom CL Equipe Set 1 batteri (GT814)/2 batterier (GT816)
- (S)** **Bruksanvisning/Teknisk Beskrivning**  
Nötkreaturs- och hästklippmaskin  
Econom CL (GT800)  
Econom CL Set 1 batteri (GT804)/2 batterier (GT806)  
Econom CL Equipe (GT810)  
Econom CL Equipe Set 1 batteri (GT814)/2 batterier (GT816)
- (FIN)** **Käyttöohjeet/Tekninen Kuvaus**  
Karjan ja hevosen klipperit  
Econom CL (GT800)  
Econom CL Set 1 akku (GT804)/2 akkuja (GT806)  
Econom CL Equipe (GT810)  
Econom CL Equipe Set 1 akku (GT814)/2 akkuja (GT816)
- (RUS)** **Инструкция по применению/Техническое описание**  
Машинка для стрижки крупного рогатого скота и лошадей  
Econom CL (GT800)  
Econom CL Set 1 аккумулятор (GT804)/2 аккумулятора (GT806)  
Econom CL Equipe (GT810)  
Econom CL Equipe Set 1 аккумулятор (GT814)/2 аккумулятора (GT816)
- (PL)** **Instrukcja użytkowania/Opis Techniczny**  
Maszynka do strzyżenia sierści bydłęcej i końskiej  
Econom CL (GT800)  
Econom CL Set 1 akumulator (GT804)/2 akumulatory (GT806)  
Econom CL Equipe (GT810)  
Econom CL Equipe Set 1 akumulator (GT814)/2 akumulatory (GT816)
- (IL)** **הוראות שימוש / תיאור טכני**  
מכונת הספורת לפרות ולסוסים  
Econom CL (GT800)  
Econom CL Set 1 Akku (GT804)/2 Akkus (GT806)  
Econom CL Equipe (GT810)  
Econom CL Equipe Set 1 Akku (GT814)/2 Akkus (GT816)

**B | BRAUN**  
SHARING EXPERTISE

## Econom CL/Econom CL Equipe cattle and horse clipping machine

### Overview

Pos.	Art. no.	Number	Name
1	GT502	1	Lower cutter plate (GT806/GT816)
2	GT501	1	Upper cutter plate (GT806)
	GT505	1	Upper cutter plate (GT816)
3	GT610405	1	Knurled nut
4	GT367205	1	Adjusting screw
6	TA010017	2	Drive pin
7	GT367201	1	Oscillator
8	GT610304	1	Oscillating block
9	GT610010	2	Fixing screw
10	GT367202	6	Oscillating spring leaf
11	GT367812	1	Spur gear with eccentric shaft
15	TA008295	2	Guide pin
16	GT367801	1	Mounted cutter head housing
17	TA005091	2	Drive pin
18	GT367802	1	Left distance part
19	GT367803	1	Right distance part
20	TA007765	1	Countersunk screw
25	GT610307	1	Spring bridge
26	GT367207	1	Fixation screw
27	GT367804	1	Mounted oscillator
35	TA009222	1	Screwdriver
36	GT800249	1	On/off button (with LED)
37	GT800804	1	Airfilter (blue) Econom CL
38	GT800844	1	Airfilter (bordeaux) Econom CL Equipe
39	GT801	1	Lithium ion battery
40	GT803	1	Charger
a	-	1	Lubricating point for battery-operated clipping machine
b	-	1	Tips of the oscillator
c	-	1	Holes in the upper cutter plate
d	-	1	Li-ion battery charging status display
ze	-	1	Li-ion battery button
f	-	1	Battery lock
t	GT800/ GT810	1	In the UK
-	GT604	1	Special lubricating oil for cutter head and cutter plates
	GT605	1	Gear grease (tube)

### Symbols on product

	Caution, general warning symbol Caution, see documentation supplied with the product
	Labeling of electrical and electronic devices according to directive 2012/19/EU (WEEE), see Disposal

### Contents

1.	Safe handling .....	3
2.	Product description .....	3
2.1	Package includes.....	3
2.2	Components required for operation .....	3
2.3	Intended use.....	3
2.4	Operating principle.....	4
3.	Preparation .....	4
4.	Working with the battery-operated clipping machine, charger and lithium ion battery .....	4
4.1	System set-up.....	4
4.2	Function check.....	4
4.3	Connecting the charger/Charging the Li-ion battery.....	5
4.4	Change the Li-ion battery .....	5
4.5	Handling the cutter head .....	6
4.6	Storage.....	7
5.	Cutter plates.....	7
5.1	Cutter plates compatible with cutter head GT367 .....	7
5.2	Recommended combinations .....	7
6.	Reprocessing .....	8
6.1	Cleaning/disinfection.....	8
6.2	Inspection and checking .....	8
7.	Maintenance .....	8
8.	Troubleshooting.....	9
8.1	Battery-operated clipping machine Econom CL/ Econom CL Equipe .....	9
8.2	Charger and Li-ion battery.....	10
9.	Technical Service.....	11
10.	Accessories/Spare parts.....	11
11.	Technical Data .....	12
11.1	Battery-operated clipping machine Econom CL/ Econom CL Equipe .....	12
11.2	Charger .....	12
11.3	Li-ion battery .....	12
11.4	Declaration of conformity .....	12
12.	Disposal.....	12
12.1	For EU countries only.....	12
12.2	Li-ion battery .....	12
13.	Two year guarantee.....	13

## 1. Safe handling

**Risk of death by electric shock!**

- Do not open the product.
- Connect the product only to a grounded power supply.

- Prior to use, check that the product is in good working order.
- Store the charger and Li-ion batteries at room temperature.
- Keep dry the charger and Li-ion batteries.
- Charge the Li-ion batteries prior to first use.
- To prevent damage caused by improper setup or operation, and to not compromise the manufacturer warranty and liability:
  - Use the product only according to these instructions for use.
  - Follow the safety and maintenance instructions.
  - Only combine Aesculap products with each other.
- Keep the instructions for use accessible for the user.
- Always adhere to applicable standards.
- Ensure that the electrical facilities in the room in which the device is being used conform to IEC norms.
- Unplug the device by pulling the plug, but never by pulling the power cord.
- Do not operate the product in explosion-hazard areas.
- Do not use the product if it is damaged or defective. Set aside the product if it is damaged.

## 2. Product description

### 2.1 Package includes

Art. no.	Designation
GT800 GT810	Battery-operated clipping machine Econom CL and Econom CL Equipe
GT801	Li-ion battery
GT803	Charger
see Accessories/Spare parts	Mains power cord (with device plug)
TA013895	Instructions for use
GT604	Hand oiler

### 2.2 Components required for operation

- Battery-operated clipping machine Econom CL/Econom CL Equipe
- Charger
- Mains power cord (with device plug)
- Li-ion battery

### 2.3 Intended use

#### **Battery-operated clipping machine Econom CL/Econom CL Equipe**

The battery-operated clipping machine Econom CL/Econom CL Equipe is used for clipping large animals such as cattle, horses and large dogs.

#### **Charger GT803/Li-ion battery GT801**

The GT803 charger is used to charge the GT801 lithium ion battery.

## Econom CL/Econom CL Equipe cattle and horse clipping machine

### 2.4 Operating principle

#### Battery-operated clipping machine Econom CL/Econom CL Equipe

The battery-operated clipping machine Econom CL/Econom CL Equipe is switched on and off using the on/off button **36**.

#### Note

The on/off button only reacts after >0.5 s in order to avoid the machine being switched on or off unintentionally.

#### Charger GT803/Li-ion battery GT801

The charger GT803 is designed for a mains voltage/frequency range of 100 V to 240 V and 50 Hz to 60 Hz.

To make the charger GT803 ready for operation, connect it to mains power by plugging in the mains plug.

The charger GT803 is equipped with one charging bay.

Charging starts automatically as soon as the Li-ion battery is inserted in the charging station.

The charging time depends on the charge state and capacity of the Li-ion battery.

#### Charging principle

The Li-ion batteries are charged by constant current pulses.

The charge state of the Li-ion battery is monitored during charging. Monitoring of the charging curve ensures that the battery is fully charged without overcharging.

Additionally, the battery temperature and charging time are monitored as well.

The charge status display **d** of the lithium ion battery has 5 LEDs. Each LED stands for 20 % of charge volume. When the battery is charging, the LED of the respective charging status blinks. When all the LEDs are permanently illuminated, the battery is completely charged.

#### Charging time

Charging is stopped as soon as the maximum charging time is reached. The charging time is approx. 70 min.

### 3. Preparation

Non-compliance with the following rules will result in complete exclusion of liability on the part of Aesculap.

- Always observe the following guidelines when setting up and operating the product:
  - national regulations for installation and operation,
  - the national regulations on fire and explosion protection,
  - operating advisories according to IEC-/VDE regulations.

### 4. Working with the battery-operated clipping machine, charger and lithium ion battery

#### 4.1 System set-up

##### Connecting the accessories

Combinations of accessories that are not mentioned in the present instructions for use may only be employed if they are specifically intended for the respective application, and if they do not compromise the performance and safety characteristics of the products.

- Please address your B. Braun/Aesculap partner or Aesculap Technical Service with any inquiries in this respect, address: see Technical Service.

#### 4.2 Function check

##### Battery-operated clipping machine Econom CL/Econom CL Equipe

- Carry out a visual inspection.
- Ensure that the airfilter **37/38** is positioned correctly, see Fig. G.
- Push the lithium ion battery **39** into the machine shaft and lock it.
- Check if the battery is charged-up: Press button **e**, see Fig. F.
- The charge status display **d** indicates the charge status.

#### Note

In order to prevent the lithium iron battery discharging when the machine is not being used, the electronic connection between the lithium ion battery and the control system for the battery-operated clipping machine is cut after 1 hour.

- Reactivating the lithium ion battery **39**: Press button **e**, see Fig. F. All the LEDs in the charge status display **d** illuminate.

##### Charger/Li-ion battery

- Carry out a visual inspection.
- Prior to connecting the product to mains power:
  - Check the mains power cord for any damage.
  - Check the product for any damage (e.g. bent contacts of the charging bay).
- Plug in mains power cord at the device socket of the charging station.
- Plug in the mains plug at the mains power socket.
- Place the lithium iron battery **39** in the charging bay of the charger and charge it, see Fig. F.

### 4.3 Connecting the charger/Charging the Li-ion battery

**Fire hazard due to battery poles shorted by fluids or metal components!**

- Li-ion battery not inserted

**Risk of injury due to damaged mains power cord!**

- Check mains power cord for any damage before charging.
- Arrange for any damaged mains power cord to be replaced by the manufacturer.

**Risk of injury and material damage due to damaged Li-ion battery!**

- Prior to charging, check the Li-ion battery for any damage.
- Do not use or charge any damaged Li-ion batteries.

**Loss of capacity/performance due to prolonged storage of the discharged Li-ion battery!**

- Store Li-ion batteries only in fully charged condition and recharge stored batteries once every month.

**Destruction of the product, charging bay or Li-ion battery when inserting the wrong type of battery in the charging bay!**

- Insert Li-ion battery only in the appropriate charging bay.

#### Note

The charger can be used by children if 8 years or older, as well as by persons with reduced physical, sensory or mental capacities or with lack of experience and knowledge, if these persons are supervised or have been instructed on how to use the device safely, and if they understand the risks associated with the use of the device.

- Do not allow children to play with the device.
- Do not allow children to clean or maintain the device without supervision.

#### Note

The Li-ion battery heats up slightly during charging.

#### Note

Disconnect the mains power cord at the mains socket when the charger is not going to be used.

- Only use the Aesculap GT803 charger for charging the lithium ion batteries.
- To connect charger 40: Plug the device plug of the mains power cord into the charger.
- Plug in the mains plug of mains power cord at the mains power socket.
- Plug the lithium iron battery 39 into the charging bay of the charger and charge it, see Fig. F.

The LEDs in the charge status display **d** of the lithium ion battery blink in accordance with the current charge status. When all the LEDs are permanently illuminated, the battery is completely charged.

### 4.4 Change the Li-ion battery

- Hold the battery-operated clipping machine **g** with the cutter head downwards so that the lithium ion battery **39** does not fall down when loosened, see Fig. 1.
- Loosen the battery lock **f** with your thumb.

Fig. 1

## Econom CL/Econom CL Equipe cattle and horse clipping machine

- Remove the lithium ion battery **39**, and push the charged replacement lithium ion battery GT801 into the battery-operated clipping machine **g**, see Fig. 2.

Fig. 2

- Secure the lithium ion battery **39** using the battery lock **f**, see Fig. 3.

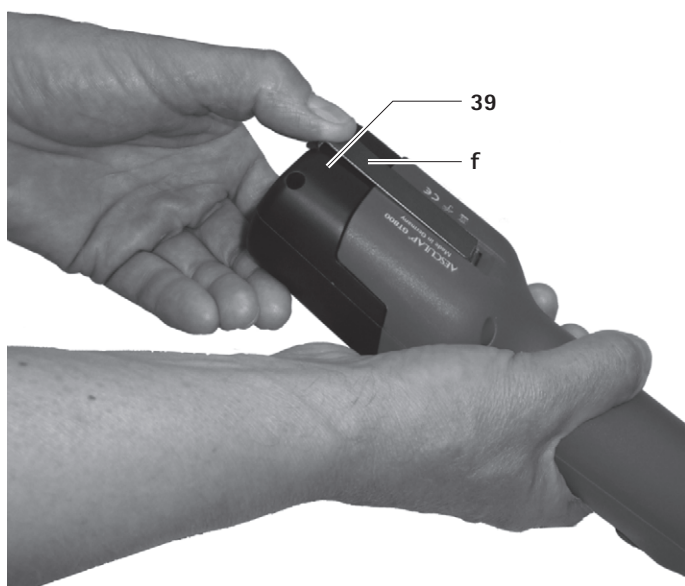
Fig. 3

### 4.5 Handling the cutter head

#### Lubricating the cutter head

**Damage to the clipping machine or the cutter head due to incorrect lubrication!**

- **Only oil the cutter head when the machine is running.**
- **Never immerse the cutter head in liquid of any kind!**
- **Only use GT604 light lubricating oil.**
- Generously oil the cutter head at the lubrication point **a** before shearing each animal, see Fig. A.
- Oil spring leaves **10 2** to 3 times daily with 3 to 4 drops of oil, see Fig. C.

#### Lubricating the gear wheel

**Damage to the clipping machine or the gear wheel due to incorrect lubrication!**

- **Use only gear grease GT605.**
- **Lubricate the gear wheel 1 to 2 times during the shearing season.**
- Unlocking and removing the lithium ion battery **39**.
- Remove both screws **6**, see Fig. B.
- Then turn the cutter head very slightly (about 15°) counterclockwise and remove it from the motor shell.
- Apply approximately 1 cm of grease on the two opposite points of the gear wheel **11**, see Fig. C.
- Carefully place the cutter head on the motor. When re-attaching the cutter head, ensure that it is turned 15° counterclockwise in such a way that its lower flange engages with the groove of the motor housing.

#### Changing the cutter plate

- Unlocking and removing the lithium ion battery **39**.
- Loosen the locking cap **3** and adjusting screw **4** by approximately 2 turns, see Fig. A.
- Loosen both clamp screws **9**, pull off the cutter plate **1** forwards and remove the upper cutter plate **2**, see Fig. D and Fig. E.
- Before inserting the upper cutter plate **2**, ensure that the oscillating block **8** is in the intended recess of the oscillator **7**, see Fig. E.
- Insert the upper cutter plate **2**. When doing so, ensure that both tips **b** of the oscillator **7** engage precisely in the holes **c** of the upper cutter plate **2**, see Fig. D.

- Push the lower cutter plate **1** below the heads of the clamping screws **9** all the way on to the cutter head shell and screw both clamping screws **9** tight.
- Tighten the adjusting screw **4** until both cutter plates are pressing slightly against one another, see Fig. A.

### Adjusting the plate pressure

#### Note

The plate pressure must be set with the machine running and without auxiliary tools.

#### Note

If the plate pressure is too high, this results in the build-up of heat and greater wear to the cutter plates.

- Carefully loosen the locking cap **3** and adjusting screw **4** until the upper cutter plate **2** is no longer pressing against the lower cutter plate **1** (the sound of the motor becomes noticeably higher-pitched), see Fig. A.
- Screw in the adjusting screw **4** until you feel slight resistance. From this point, keep turning approximately one full turn and then tightened the locking cap **3**.

### Cleaning the cutter head

- The cutter head must be cleaned at least after each day of clipping time or if it gets dirty.
- Unlocking and removing the lithium ion battery **39**.
- Remove both screws **6**, see Fig. B.
- Then turn the cutter head very slightly (about 15°) counterclockwise and remove it from the motor shell.
- Remove the cutter plates, see Changing the cutter plate.
- Thoroughly clean the interior part and individual parts of the cutter head using a brush.
- Insert the cutter plate and tighten, see Changing the cutter plate.
- Lubricate the gear wheel, see Lubricating the gear wheel.
- Carefully place the cutter head on the motor. When re-attaching the cutter head, ensure that it is turned 15° counterclockwise in such a way that its lower flange engages with the groove of the motor housing.
- Lubricate the cutter head, see Lubricating the cutter head.

### Cleaning the air filter

- Clean the airfilter **37/38** at least once per day or if it gets dirty.
- Remove the airfilter **37/38** from the battery-operated clipping machine, see Fig. G.
- Thoroughly clean the airfilter **37/38** in warm water with a brush.
- Leave the airfilter **37/38** to dry.
- Clean the battery-operated clipping machine.
- Place the airfilter **37/38** on the battery-operated clipping machine.

### Re-sharpen the cutter plates.

- In order to re-sharpen the cutter plates, contact Aesculap Suhl Technical Service, see Technical Service.

## 4.6 Storage

- Clean the clipping machine and cutter plates thoroughly before storing, see Reprocessing.
- Store the clipping machine in a dry, dark and clean room.

## 5. Cutter plates

### 5.1 Cutter plates compatible with cutter head GT367

Art. no.	Designation
GT501	Upper cutter plate with 15 cutting teeth
GT502	Lower cutter plate with 31 cutting teeth, Cutting height: 3 mm
GT503	Upper cutter plate with 17 cutting teeth
GT504	Lower cutter plate with 18 cutting teeth, Cutting height: 3 mm
GT505	Upper cutter plate with 23 cutting teeth
GT506	Lower cutter plate with 23 cutting teeth, Cutting height: 3 mm
GT508	Lower cutter plate with 51 cutting teeth, Cutting height: 1 mm
GT510	as GT506, but a cutting height of 5 mm
GT511	as GT502, but a cutting height of 1 mm

### 5.2 Recommended combinations

Combination	Upper/lower plate	Suitable for
closely spaced teeth	GT501/GT502	Cattle and horses (regular equipment for GT474 and GT367)
	GT501/GT511	Industrial shearing
	GT505/GT508	(e.g. coat shearing)
medium spaced teeth	GT505/GT506	Cattle, dogs and goats
	GT505/GT510	Dogs and goats
widely spaced teeth	GT503/GT504	Cattle, sheep with fine wool and single sheep

## Econom CL/Econom CL Equipe cattle and horse clipping machine

### 6. Reprocessing

#### 6.1 Cleaning/disinfection

**DANGER**

**Risk of electric shock and fire hazard!**

- **Before cleaning:**
  - Remove the mains plug from the charger.
  - Unlock the lithium ion battery and remove from the battery-operated clipping machine.
- **Do not use flammable or explosive cleaning or disinfecting solutions.**
- **Ensure that no fluids will penetrate the product.**

**CAUTION**

**Damage to, or destruction of the product caused by mechanical cleaning/disinfection!**

- **Only clean and disinfect the product manually.**
- **Do not sterilize the product under any circumstances.**

**CAUTION**

**Risk of damage to the product due to inappropriate cleaning/disinfecting agents!**

- **Only use cleaning/disinfecting agents approved for surface cleaning. Follow the manufacturer's instructions for the respective cleaning/disinfecting agent.**
- **Never immerse the product in water or detergent.**
- **Do not introduce any fluid into the charging bays.**
- **Clean the contacts in the charging bay, applying extreme care.**

- Wipe the shell of the product with a lint-free cloth moistened with a commercial disinfectant for wipe disinfection.
- If necessary, after the appropriate exposure time, wipe off any residues of the cleaning/disinfecting agent with a lint-free cloth moistened with clean water.
- Dry the product with a clean, lint-free cloth.
- Clean the contacts in the charging bay with isopropanol or ethyl alcohol and a cotton swab. Do not use corrosive chemicals.
- If necessary, repeat the cleaning/disinfecting process.

#### 6.2 Inspection and checking

- Allow the product to cool down to room temperature.
- Inspect the product after each cleaning and disinfecting cycle to be sure it is: clean, functional, and undamaged.
- Check the product for any damage, abnormal running noise, overheating or excessive vibration.
- Inspect the shearing head for broken, damaged or blunt cutting edges.
- Set aside the product if it is damaged.

### 7. Maintenance

To ensure reliable operation, Aesculap recommends servicing the product **at least once a year**.

For technical service, please contact your national B. Braun/Aesculap agency, see Technical Service.

## 8. Troubleshooting

### 8.1 Battery-operated clipping machine Econom CL/Econom CL Equipe

Malfunction	Detection	Cause	Remedy
The battery-operated clipping machine does not start	-	Li-ion battery not activated	Press button e on the lithium ion battery
	The LED of the on/off button <b>36</b> blinks twice per second	Li-ion battery not loaded	Load Li-ion battery
	-	Li-ion battery defect	Insert new Li-ion battery
	-	Li-ion battery not inserted	Insert the Li-ion battery
The battery-operated clipping machine pauses during operation	-	Li-ion battery deactivated through protective circuit	Switch the battery-operated clipping machine off and on again
	-	Li-ion battery is empty	Insert new Li-ion battery

Econom CL/Econom CL Equipe cattle and horse clipping machine

8.2 Charger and Li-ion battery

Malfunction	Detection	Cause	Remedy
Charger not working	The "Battery" LED indicator is not lit up	Mains power cord not plugged in	Plug mains power cord into the device socket on the charger and into the socket of the mains power supply
	Li-ion battery not loaded	Mains power cord defective	Replace mains power cord
		Li-ion battery defect	Send Li-ion battery to manufacturer for repair, see Technical Service
Li-ion battery cannot be loaded	Li-ion battery plugged in, charge state indicator does not light up	Charger contacts dirty	Clean the contacts in the charging bay, see Reprocessing
	-	Charger contacts damaged	Send charger to manufacturer for repair, see Technical Service
	-	Li-ion battery defect	Send Li-ion battery to manufacturer for repair, see Technical Service
	-	Charger defect	Send charger to manufacturer for repair, see Technical Service
	-	Too high a temperature is measured in the lithium ion battery during the charging process	Remove the Li-ion battery from the charging bay, leave to cool and repeat the charging process
			If the problem occurs again: Send Li-ion battery to manufacturer for repair, see Technical Service
-	Battery charging error or Li-ion battery defect	Remove the Li-ion battery from the charging bay and repeat the charging process	
		If the problem occurs again: Send Li-ion battery to manufacturer for repair, see Technical Service	

## 9. Technical Service

**Risk of injury and/or malfunction!**

➤ **Do not modify the product.**

➤ For service and repairs, please contact your national B. Braun/Aesculap agency.

Modifications carried out on medical technical equipment may result in loss of guarantee/warranty rights and forfeiture of applicable licenses.

### Service addresses

Aesculap Suhl GmbH

Fröhliche-Mann-Straße 15

98528 Suhl / Germany

Phone: +49 3681 4982-0

Fax: +49 3681 4982-34

Email: [suhl@aesculap.de](mailto:suhl@aesculap.de)

[www.aesculap-schermaschinen.de](http://www.aesculap-schermaschinen.de)

Other service addresses can be obtained from the address indicated above.

## 10. Accessories/Spare parts

Art. no.	Designation	Design	Power cord or power supply unit	Li-ion battery	Complete charging station	Accessories
GT804/ GT806	Econom CL	Europe outside of Great Britain	TA012170 European plug, black L=1.8 m	GT801	GT803	GT604 Special lubricating oil for cutter plates and cutter head  GT605 Gear grease (tube)
GT814/ GT816	Econom CL Equipe					
GT804G/ GT806G	Econom CL	Great Britain	TA012169 GB plug, black, L=1.8 m			
GT814G/ GT816G	Econom CL Equipe					
GT804K/ GT806K	Econom CL	USA	TA012168 2-pin connector plug, black, L=1.8 m			
GT814K/ GT816K	Econom CL Equipe					
GT804A/ GT806A	Econom CL	Australia	TA013657 2-pin connector plug, black, L=1.8 m			
GT814A/ GT816A	Econom CL Equipe					

## Econom CL/Econom CL Equipe cattle and horse clipping machine

### 11. Technical Data

#### 11.1 Battery-operated clipping machine Econom CL/ Econom CL Equipe

	GT800 Econom CL	GT810 Econom CL Equipe
Stroke rate	max. 2 750 1/min	max. 2 250 1/min
Running time with a lithium ion battery	approx. 70 min	approx. 80 min
Nominal voltage	max. 21.6 V	max. 21.6 V
Capacity	max. 2.6 A	max. 2.6 A
Weight (with shearing head and battery)	1 250 g	1 250 g
Quality marks	TÜV-GS, CE, UL, CSA	TÜV-GS, CE, UL, CSA

#### 11.2 Charger

	GT803
Voltage range (current consumption)	100 V to 240 V
Frequency	50 Hz to 60 Hz
Charging voltage/Output voltage	max. 25.2 V
Charging current/Output current	max. 2.6 A
Weight (with cable and power supply unit)	764 g
Quality marks	TÜV-GS, CE, UL, CSA

#### 11.3 Li-ion battery

	GT801
Cell type	Li
Nominal voltage	21,6 V
Capacity	2,6 Ah
Charging time	approx. 70 min
Quality marks	TÜV-GS, CE, UL, CSA

### 11.4 Declaration of conformity

	<p>We declare under our sole responsibility that this product complies with the following standards and regulatory documents:</p> <p>2004/108/EC EMC Guidelines: DIN EN 60335-1 DIN EN 60335-2-8 Safety of household and similar electrical appliances</p> <ul style="list-style-type: none"> <li>- general requirements</li> <li>- for hair shearing machines</li> </ul> <p>Aesculap Suhl GmbH</p>
---	---

### 12. Disposal

Electrical tools, accessories and packaging should be sent for environmentally responsible recycling.

#### 12.1 For EU countries only

	<p>Do not dispose of electrical tools in domestic waste! According to European Directive 2012/19/EU on old electrical and electronic appliances and its implementation in national law, electrical tools that are no longer suitable for use must be separately collected and sent for environmentally responsible recycling.</p>
---	---

- If you have any questions concerning the disposal of the product, please contact your national B. Braun/Aesculap agency, see Technical Service.

#### 12.2 Li-ion battery

Do not dispose of lithium ion batteries in domestic waste, fire or water. Defective or used lithium ion batteries must be collected, recycled or disposed of in an environmentally-friendly way according to Directive 2006/66/EC.

- If you have any questions concerning the disposal of the product, please contact your national B. Braun/Aesculap agency, see Technical Service.

## 13. Two year guarantee

Dear Customer,

Many thanks for choosing one of our products.

For decades, the name Aesculap has stood for proven quality and first-class service. We are known to provide innovative, high-performance products for our customers. Aesculap manufactures high-quality devices and can assure you that it uses high-quality materials and careful manufacturing processes. We would like to point out to you that we are not liable for defects in our products, if these are caused by improper use, normal wear or are caused by the product being improperly used or if inappropriate or not properly functioning third-party products are used.

Parts that are typically subject to wear, and for which we are therefore not liable, are the battery and shearing head. Material defects that do not affect or only negligibly affect the value or function of the product are also excluded from our warranty.

If we are liable under our warranty, we reserve the right to repair or replace the product.

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